

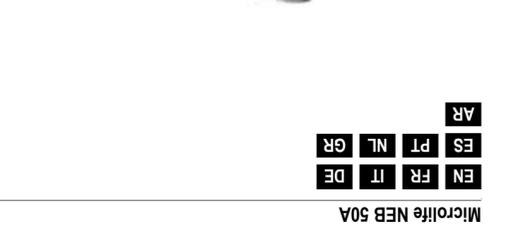
microlife.com
 F. Fax +1 721 442 5377
 T. Tel. +1 721 442 5353
 COAHAMAR, FL 32755 USA
 L1511 GULF RD, BLD 202, FLOOR 512
 MICROLIFE USA, INC.
 NORTH/CENTRAL / SOUTH AMERICA
 microlife.com
 F. Fax +882 2 8797 1283
 T. Tel. +882 2 8797 1288
 L1511 GULF RD, BLD 202, FLOOR 512
 MICROLIFE SOUTH AMERICA
 microlife.com
 F. Fax +1 771 777 77 30
 T. Tel. +1 771 777 77 30
 9442 WINDEN / WINDEN
 ELSBACHEN 136
 MICROLIFE EUROPE / MIDDLE-EAST / AFRICA
 Distributor

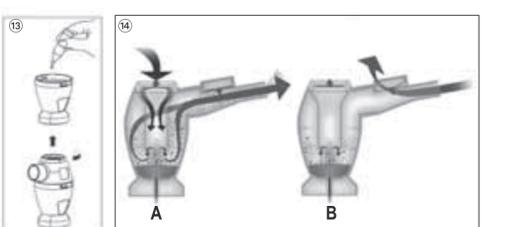NEB 50A
 IEB NEB 50A-V-4515

microlife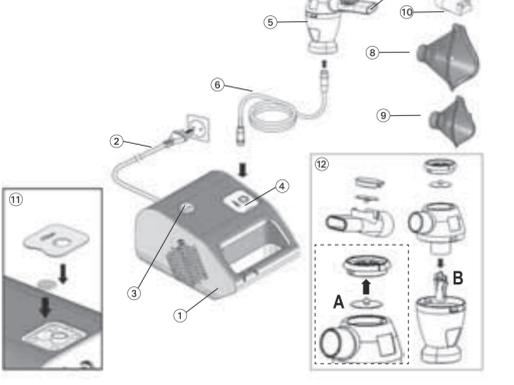

Guarantee Card
 Name of Purchaser / Nom de l'acheteur / Nome del rivenditore / Name des Käufers / Nombre del comprador / Nome do comprador / Naam koper / Ονομαστού αγοραστή / اسم المشتري
 Serial Number / Numéro de série / Numero di serie / Serien-Nr. / Número de serie / Número de série / Seriennummer / Αριθμός σειράς / رقم التسلسل
 Date of Purchase / Date d'achat / Data di acquisto / Kaufdatum / Fecha de compra / Data da compra / Datum van aankoop / Ημερομηνία αγοράς / تاريخ الشراء
 Specialist Dealer / Revendeur / Categoria rivenditore / Fachhändler / Vendedor especializado / Revendedor autorizado / Spécialist Dealer / Εξειδικευμένος αντιπρόσωπος / متخصص الموزع
 3A Health Care S.r.l.
 Via Marziale Carutti Loc. San Tomaso
 25017 Lonato (BS) – Italy
 Tel. 0039 030 9133177
 Fax 0039 030 9919114

CE0434

Compressor Nebuliser

- Piston compressor
- Power lead
- ON/OFF Switch
- Air filter compartment
- Nebuliser «NEB PRO»
- Air hose
- Mouthpiece with valve
- Adult face mask
- Child face mask
- Nose piece
- Replacing air filter
- Assembling nebuliser kit
- Filling the nebuliser
- Breath-enhanced nebulisation

Dear Customer,
 This nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses.

The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your MicroLife product. If you have any questions, problems or want to order spare parts please contact your local MicroLife-Customer Service. Your dealer or pharmacy will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – MicroLife AG!

- Read the instructions carefully before using this device.
- Type BF applied part
- Keep dry

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Never bend the nebuliser over 90°.
- The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please notify the manufacturer properly before use.
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Use of this device is not intended as a substitute for a consultation with your doctor.

7. Garantie
 This device is covered by a 3 year guarantee from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.
 • The guarantee covers only the compressor. The replaceable components like nebuliser, masks, mouthpiece, air hose, and filters are not included.
 • Opening or altering the device invalidates the guarantee.
 • The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

8. Technical Specifications

Nebulisation rate:	0.5 ml/min. (NaCl 0.9%)
Particle size:	75% < 5 µm NaCl 0.9% 2.85 µm (MMMD with 0.9% NaCl using Cascade Impactor)
Compressor air flow:	13 l/min.
Acoustic noise level:	53 dBA
Power source:	230V - 50Hz / 150VA
Power lead length:	1.5 m
Nebuliser capacity:	min. 2 ml; max. 12 ml
Residual volume:	0.7 ml
Operating limits:	30 min. On / 30 min. Off
Operating conditions:	10 - 40 °C / 50 - 104 °F 10-95 % relative maximum humidity
Storage conditions:	-25 - +70 °C / -13 - +158 °F 10-95 % relative maximum humidity
Weight:	1300 g
Dimensions:	209 x 140 x 124 mm
IP Class:	IP20
Reference to standards:	EN 13544-1; EN60601-1; EN 60601-1-2; IEC 60601-1-11; CE0434
Expected service life:	5 years

Technical alterations reserved.

2. Preparation and Usage of this Device

- Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».
- Assemble the nebuliser kit (2). Ensure that all parts are complete.
 - Fill the nebuliser (3) with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum volume.
 - Connect the nebuliser with the air hose (4) to the compressor (1) and plug the power lead (2) into the socket (230V - 50Hz).
 - Switch the ON/OFF switch (3) to position «I» to turn on the device and place the mouthpiece (7) in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between adult (8) or child face mask (9) and make sure that it encloses the mouth and nose area completely.
 - Use all accessories including the nose piece (10) as prescribed by your doctor.
 - Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell.
 - After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (3) to position «0» to turn off the device and unplug it from the socket.
 - Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

This device was designed for intermittent use of 30 min. On / 30 min. Off. Switch off the device after 30 min. use and wait for another 30 min. before you resume treatment.

3. Nebuliser «NEB PRO» treatment options

This device is equipped with the innovative nebuliser «NEB PRO» (5) which includes the breath-enhanced valve system (14). All common medication for inhalation therapy can be nebulised effectively.

Advanced treatment
 This treatment option allows the maximum amount of medication to be delivered thus reaching the respiratory capacity of each patient. The valve system optimally adapts the flow of medication during inhalation (14-A), reducing medication loss during exhalation (14-B). Use the breath-enhanced valve system (synchronized with breathing) to optimize the therapeutic effects of the medication and to get the maximal benefit.

- Mount the membrane on the valve-holder and use the mouthpiece with expiratory valve (12-A).
- Follow the usage steps as described in «Section 2.»

Only using the mouthpiece with expiratory valve (12) enables the synchronized functioning of the nebuliser.

Short treatment
 The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.

- A synchronized nebulisation is not possible using this treatment option.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor (1) and the air hose (6) should be cleaned with a clean, moist cloth.

Always wash your hands well before cleaning and disinfecting the accessories.

- Do not expose the compressor to water or heat.
- Replace the air hose for each treatment with a new patient or in case of impurities.
- Unplug the device before cleaning.

Cleaning with water
 Wash all nebuliser components (except air hose) under warm tap water (max. 60 °C) for about 5 minutes adding if necessary a small quantity of

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